

Cleaning Your Loupes

Do's and Don'ts

Warning! Improper cleaning methods can damage your loupes. Please follow these instructions carefully to ensure proper cleaning.

Don't



Don't rinse loupes under running water.



Avoid sterilizing in autoclaves or chemiclave machines.



Don't submerge in any liquid or ultrasonic cleaner.



Don't use Lysol® or any disinfectants containing glutaraldehydes or iodophors.




Don't use cleaning materials that are based ammonia, bleach and acids.

admetec
We've got your back



Do

Before cleaning, ensure to unplug and detach any wires and cables.

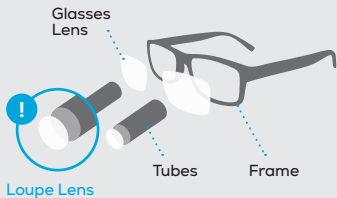

To Clean Your Loupes:



Use eyeglass cleaners or a mild soap-and-water solution with a soft cloth. After cleaning, dry with a soft cloth or air dry.

Be extra gentle when cleaning the loupes lenses to avoid causing scratches

- * If cleaners leave residue on the lenses, use a soft, lint-free cloth to remove it.

To Disinfect Your Loupes:



Use OPTIM 1 Wipes, Caviwipes or Admetec cloth with any alcohol up to 70%.



Wipe all exposed plastic surfaces, including frame, temples and tubes.



Dry with a soft cloth or air dry.

admetec
We've got your back

